

Order of Draw Lab phone: 283-7090		
Blood Cultures	SDS	Green first then Red top, Peds 1-5 ml
Blue Top	Na Citrate	Coagulation - Fill to line
GOLD /SST	Clot Tube	Chemistry/Serology - Allow 20 min to clot
Green Top	Lithium Heparin	Chemistry - For ammonia testing- on ice to lab ASAP.
Lavender	EDTA	Hematology – always mix 5-8 times after draw
Pink TOP	EDTA	Blood Bank – Type and Screen, Xmatch
Grey Top	Na Fluoride	Lactic Acid - Fill at least half full
*Label: Pt Name, DOB, time, date, Meditech mnemonic		