
Patient collection procedure guide

For vaginal swab specimens

Kit includes: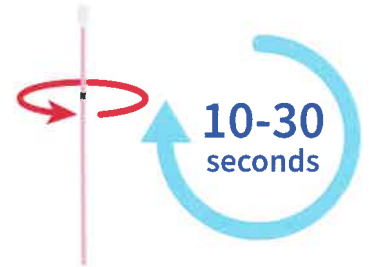


1

Wash hands before starting. If you have any questions about this procedure, please ask your healthcare provider.

Partially open swab package and remove swab. **Do not touch the soft tip or lay the swab down.** If the soft tip is touched, laid down, or dropped, discard and get a new Aptima Multitest Swab Specimen Collection Kit. **Hold swab, placing thumb and forefinger in the middle of swab shaft over score line.**

2

Carefully insert swab into opening of the vagina, about 2 inches (5 cm), and gently **rotate swab for 10 to 30 seconds**. Make sure swab touches the vaginal walls so that moisture is absorbed by the swab. Withdraw swab without touching skin.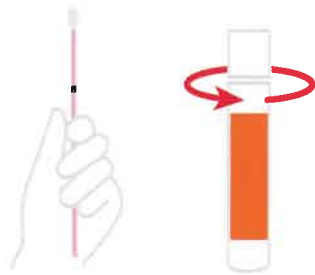


3

While holding swab in your hand, unscrew tube cap. Do not spill tube contents. If tube contents are spilled, request a new Aptima Multitest Swab Specimen Collection Kit.

4

Immediately place swab into transport tube so score line is at top of tube. Align score line with top edge of tube and carefully break swab shaft.

Note: Do NOT insert score line below top edge of vial. Do NOT push swab to bottom of vial.

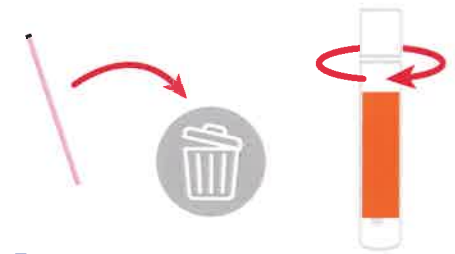5

Discard top portion of shaft. Tightly screw cap onto tube. Return tube as instructed by your healthcare provider.

To see a video, scan QR code:

or go to: labcorp.com/sti-self-collection-IFU

Instructions authored by Hologic, Inc. Images used with permission from Hologic, Inc. Hologic, Aptima and associated logos are trademarks and/or registered trademarks of Hologic, Inc. and associated companies in the United States and/or other countries.

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©2025 Labcorp. All rights reserved.